

Disposable Respirator Fitting Instructions

Moldex offers up to five sizes and five styles to fit today's diverse workforce.

MOLDEX[®]
Ideas that wear well.[®]

2-Strap



1. Hold respirator in hand with molded nose contour (narrow end) at finger tips, allowing headstraps to fall below hand.



2. Place respirator under chin with molded nose contour (narrow end) up. Nose cushion must not be creased inside of respirator. Raise top strap to top back of head. Pull shorter bottom strap over head, below ears, to around neck. Do not wear with only one strap. When changing from any model/size to another respirator, you must fit test.



3. Adjust respirator for comfortable fit.



4. Each time before entering a contaminated area, perform a user seal check. Cover front of respirator by cupping both hands. **INHALE SHARPLY.** A negative pressure should be felt inside respirator. If any leakage is detected at respirator edges, adjust straps by pulling back along the sides and/or reposition respirator. Repeat until sealed properly, otherwise see your supervisor. Entry into a contaminated area with an improper fit may result in sickness or death.